EN

Intended use
Symphony PLUS is a program card for use with the Symphony breast pump to generate pump suction patterns.

Contraindications
There are no known contraindications.


Assembly instructions
I Remove the protector for the card and cord on the back of the breast pump, if attached.
I Insert the program card into the card compartment.
I Mark the pump with the instructions sticker.
I Attach the protector for the card and cord, if available.

Program selection

The program card contains two pump programs. The goal of the **INITIATE** program is to initiate milk production. The **MAINTAIN** program is designed to build and maintain milk production.

Use the **INITIATE** program directly after birth: I Until the mother has pumped at least 20 ml in total in each of her last three pumping sessions, **OR** I Throughout the first five days. If milk production has not started after five days, change to the **MAINTAIN** program.

Use the **MAINTAIN** program after secretory activation (milk coming in):
I Once the mother has pumped at least 20 ml in each of her last three pumping sessions, **OR** I Beginning on day 6, whichever of the two options comes first.

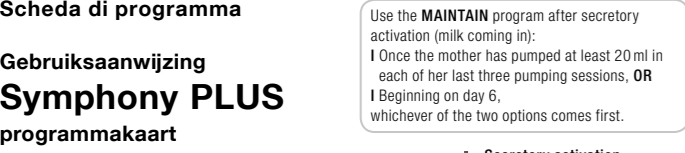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