Instructions for use Nipple Formers

Brugsanvisning Vårformare

Brugsanvisning Brystvorteformere

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Brugsanvisning Brystvorteformer

ENGLISH

Nipple Formers

The Nipple Former prepares flat or inverted nipples for breast-feeding. Flat or inverted nipples can make breastfeeding difficult. Because some babies have trouble latching correctly, which can lead to causing sore nipples. Gentle pressure on the areola from the silicone membrane gradually forces the nipple out. The therapy can be started during pregnancy (from about 32nd week onwards). Start wearing the Nipple Former for one hour a day and gradually increase to approximately 8 hours daily. After birth, use the nipple former between feedings (1/2–1 hour before breast-feeding). Avoid wearing at night to prevent tender spots. The ventilation holes in the top of the shells allow air to circulate, thus preventing humidity and promoting healing.

Assembly and Use
Inspect before each use. Replace product at first signs of damage, mold, or weakness. Put the silicone membrane on the shell. Be sure it snaps on all the way around. Place the assembled Nipple Former inside your bra with the opening in the silicone membrane centered over your nipple. The ventilation holes in the shell must point upwards. We recommend wearing a nursing bra when using a former; nursing bras are roomier and the former will fit more comfortably. Refer to a lactation consultant to ensure the pressure is adequate while remaining comfortable.
Caution: Do not feed the breast milk collected in the shell to the baby. Discard this milk.

Cleaning Instructions
Before first use and subsequently once daily
Place the separated parts in a small pan filled with cold water. Dis-filled water is recommended – if tap water is used, a teaspoon of citric acid may be added to the water to avoid a build-up of limescale. Bring the water to boil and allow it to continue boiling for approximately 5 minutes. Drain off the water and place the parts on a clean cloth to dry.

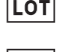
After each use
Separate the parts. Shells and membranes should be washed with a mild cleaning solution, rinsed thoroughly in cold water and dried on a clean cloth.

Note
Shells and membranes can be put in a dishwasher or autoclave.

Material
Shell: Polypropylene / Silicone membrane: Silicone

Disposal
According to local regulations.

Meaning of Symbols

| | |
|---|---|
|  | Indicates conformity with the EU’s requirements for medical devices |
|  | Indicates the manufacturer (ISO 15223-1, 5.1.1) |
|  | Indicates the date of manufacture (ISO 15223-1, 5.1.3) |
|  | Identifies a medical device |
|  | Indicates the manufacturer’s batch code or lot (ISO 15223-1, 5.1.5) |
|  | Indicates the manufacturer’s product number (ISO 15223-1, 5.1.6) |
|  | Keep away from sunlight (ISO 15223-1, 5.3.2) |
|  | Fragile (ISO 15223-1, 5.3.1) |
|  | Keep dry (ISO 15223-1, 5.3.4) |
|  | Consult the instructions for use (ISO 15223-1, 5.4.3) |

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-  AB tıbbi cihaz gerekliliklerine uygunluğu gösterir