therashells™

Breastshells
Boucliers
Copas para senos

- ✓ Instructions for use
- ✓ Instructions
- ✓ Instructivo de uso

www.medela.com

Medela, Inc.,
P.O. Box 660, 1101 Corporate Drive
McHenry, IL 60051-0660, USA
Ph: (800) 435-8316 or (815) 363-1166
Fax: (815) 363-1246
Email: customer.service@medela.com

Medela Canada, Inc.,
4160, Sladeview Crescent, #8
Mississauga, Ontario, Canada L5L 0A1
Tél.: 1-905-608-7272
Fax: 1-905-608-8720
Email: info@medela.ca

Printed in the USA. Medela is a registered trademark of Medela and TheraShells, Quick Clean and Micro-Steam are trademarks of Medela. All rights reserved.

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1907683 C 0210 © 2010 Medela, Inc.

TheraShells™ Breastshells

This product is for use with flat and inverted nipples as well as engorgement. It is important to consult with your healthcare provider or breastfeeding specialist to educate yourself on breastfeeding preparation before your baby is born and breast care while you are breastfeeding. Call 1-800 TELL YOU to locate a breastfeeding specialist in your area.

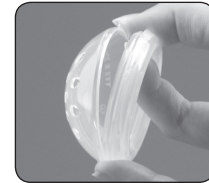
Instructions

Cleaning


Before first use, disassemble all parts and sanitize by boiling for 10 minutes. After each use, disassemble and wash all parts with mild soap and warm water, rinse thoroughly and dry completely. Plastic shell parts are safe to autoclave, boil, dishwasher and clean with Medela Quick Clean™ Micro-Steam™ bags.

IMPORTANT: When boiling or autoclaving, clean all six pieces at the same time. Disassemble prior to boiling or autoclaving.

To Disassemble

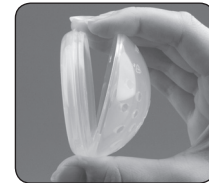Push the tab with your thumb while pulling the back with your index finger.

Or
Squeeze the assembled pieces. The back will separate from the dome.

Flat or Inverted Nipples

1. Assemble the dome and back with small hole (cone) together to form a shell.
2. Place the shell inside your bra with the cone centered over the nipple. The cone will create slight pressure at the base of the nipple, drawing it out.
3. Wear the shells for half an hour before breastfeeding to help draw the nipple out, making it easier for baby to latch on. If discomfort occurs, discontinue use.

For use with flat or inverted nipples during pregnancy, begin wearing the shells during the last couple of weeks of pregnancy.

If you are breastfeeding, only wear the shells in between feedings, during waking hours.

Engorgement

1. Assemble the dome and back with small hole (cone) together to form a shell.
2. Place the shell inside your bra with the cone centered over the nipple. If leakage occurs, a bra pad can be placed directly in your bra at the lower edge of TheraShell dome. The vents in the dome allow free circulation of air around the nipple. The shell also keeps clothing and wet bra pads off the nipple.
3. Wear the shells for half an hour before breastfeeding to help draw the nipple out, making it easier for baby to latch on. If discomfort occurs, discontinue use.

Sore Nipples

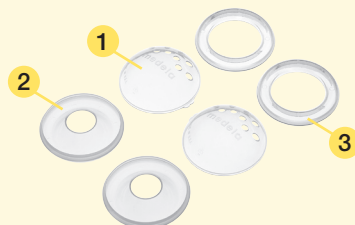
1. Assemble the dome and back with large hole (ring) together to form a shell.

2. Place the shell inside your bra with the ring centered over the nipple. If leakage occurs, a bra pad can be placed directly in your bra at the lower edge of TheraShell dome. The vents in the dome allow free circulation of air around the nipple. The shell also keeps clothing and wet bra pads off the nipple to help prevent further irritation.
3. Wear the shells during waking hours to help treat the soreness.

Ordering Information

To order parts for your TheraShells, contact your local authorized Medela location.

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