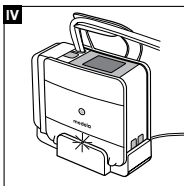
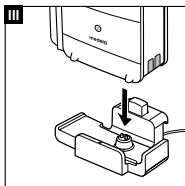
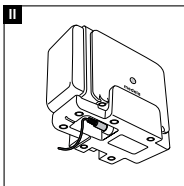
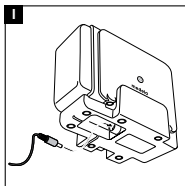
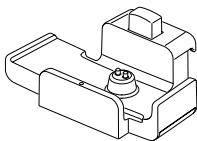
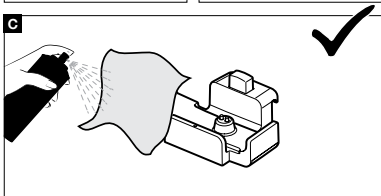
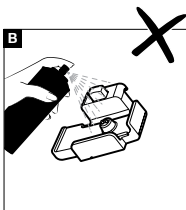
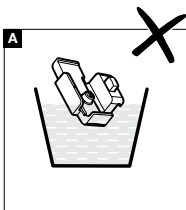
Preparation for use

Cleaning Guideline

Preparation for use / Cleaning Guideline

Cleaning and reprocessing instructions

Reusable devices and components from Medela are delivered non-sterile and are intended for reuse. Prior to use, clean and reprocess the product following the procedure described below.

The following cleaning process has been validated according to international standards and guidelines.

It is the responsibility of the user/processor to ensure that the cleaning/reprocessing procedure is performed using equipment, materials and personnel which are suitable to ensure the effectiveness of the process. Any deviation from the following instructions should be validated by the user/processor to ensure effectiveness of the process.

Materials and equipment

- Personal Protective Equipment (PPE: disposable gloves, proper protective gear)
- Lint free nonwoven wipes
- Water (<40 °C, <104 °F)

NOTES

- Disconnect device from the mains before cleaning/disinfection.
- Never place the device in water or other liquids.
- Never use steel brushes or steel wool for cleaning.
- Water temperatures higher than 45 °C (113 °F) can cause protein coagulation which may negatively affect the efficacy of the process.

Initial cleaning: Wipe the device surface using a lint free nonwoven wipe moistened with water (<40 °C, <104 °F).

Thorough cleaning: Use a lint free nonwoven wipe moistened with water (<40 °C, <104 °F).

Manual disinfection: For manual disinfection, use only alcohol-based disinfectant wipes. For effective and safe disinfection please follow the instructions provided by the disinfectant manufacturer.

Drying: After the manual disinfection, store the product on a clean dry cloth and let it dry. Alternatively, follow the instructions provided by the disinfectant manufacturer.

Inspection: After reprocessing according to the steps described above, the device and its components must be inspected for signs of degradation that may limit the useful life and / or performance of the device, such as the following: visible corrosion, mechanical wear, abrasion, damage, or deformation. Discard the device if any signs of degradation are evident.

Storage: Store the product dry and dust-free.

For further information on use and performance of Docking Station, please refer to Instructions for Use for Thopaz⁺ and Invia Liberty pumps.

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Medela AG
Medela UK Ltd.
Medela LLC