Invia[®] Motion[™]

NEGATIVE PRESSURE WOUND THERAPY SYSTEM

- EN Patient instructions for use
- DE Gebrauchsanweisung für Patienten
- FR Mode d'emploi pour le patient
- ES Instrucciones de uso para el paciente
- NL Gebruiksaanwijzing voor patiënten
- SV Bruksanvisning för patienter
- IT Istruzioni per l'uso per il paziente
- FI Käyttöohjeet potilaalle
- PT Instruções de utilização para o doente
- DA Brugsanvisning til patienter
- NO Bruksanvisning for pasient
- SK Návod na používanie pre pacientov

Assistance

Please read the entire instructions for use before trying to operate this device.

If you have any questions about your Invia Motion Negative Pressure Wound Therapy (NPWT) system, please contact your healthcare professional at the number below:

Healthcare professional contact information:

Please keep the patient instruction for use in an easily accessible location.

Table of contents

Warnings and safety instructions	4
General warnings	4
Purpose of the device	6
Intended use	6
When to use the device (Indications for use)	7
When not to use device (Contraindications)	7
Description of the device	8
Display	9
Conditions that affect use	9
General safety at home	10
Sleeping	11
Showering, bathing and washing	11
When you leave the house	12
Set up instructions	12
Set up carrying case	14
Charge battery	15
Change canister/tubing set	16
Notifications and troubleshooting	19
Info used time	26
Cleaning/Disinfection	27
Warranty	27
Maintenance	27
Disposal	27
Technical specifications	28
Signs and symbols	29
Electromagnetic compatibility (EMC)	31

Warnings and safety instructions

WARNINGS

Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTIONS

Indicates a potentially hazardous situation which, if not avoided, could result in minor or moderate injury.

Safety related tip

Indicating useful information about the safe use of the device.

General warnings

WARNING

The individual responsible for operating this device on a daily basis (patient, caregiver) must be able to read, understand and follow the directions provided in this instructions for use manual, on the Invia Motion NPWT pump display and by the healthcare professional. The individual must be able to hear and respond to the auditory signals.

Only use the Invia Motion NPWT system on the person for whom it was ordered and only for its intended use.

Do not perform the therapy without your healthcare provider's supervision.

WARNING

Visually check the wound dressing and the Invia Motion NPWT pump frequently.

If you see large amounts of blood in the dressing, tubing or canister:

1. Turn off the Invia Motion NPWT pump by pressing the  button and holding for 3 seconds.
2. Apply pressure to your wound.
3. Seek immediate emergency medical assistance.

If you have difficulty breathing, seek immediate emergency medical assistance.

 **WARNING**

Immediately contact your healthcare professional if you experience any of the following symptoms:

- Significant vision or hearing loss
- Pain, soreness or redness around the wound
- Swelling, itching or rash around the wound
- Fever
- Strong odor from the wound
- Thick green drainage or any change in color of drainage
- Nausea, vomiting or diarrhea
- Difficulty breathing
- Dizziness or confusion
- Allergic reactions like redness, rash or hives, swelling or severe itching

 **WARNING**

Once the healthcare professional has set the Invia Motion NPWT pump pressure and mode settings, do not press the pressure  or mode  setting buttons.

Pressing these buttons may change the settings of the Invia Motion NPWT pump and could cause harm.

 **WARNING**

The Invia Motion NPWT pump must be used 24 hours per day. Do not stop the Invia Motion NPWT pump unless instructed by your healthcare professional or if excessive bleeding occurs. If the Invia Motion NPWT pump is stopped for more 2 hours, the dressing must be changed. Contact your healthcare professional to request a dressing change.

WARNING

If the Invia Motion NPWT pump stops working and cannot be switched on, immediately call your healthcare professional.

Purpose of the device

Your healthcare provider has prescribed Negative Pressure Wound Therapy (NPWT) for you.

Negative Pressure Wound Therapy has been shown to help promote healing in several different kinds of wounds. Negative pressure (suction) is delivered to a wound through a pump. The suction, or negative pressure, helps to remove fluid and close your wound. Your healthcare professional will place a special dressing on your wound and a tube will go from your wound to the canister on the pump. The pump turned on and set to the prescribed pressure to remove the fluid from your wound. The fluid will be collected in the canister. The treatment will be applied 24 hours a day.

Intended use

The Invia Motion NPWT system is intended to be used by healthcare professionals or adequately trained lay users.

Healthcare professionals are responsible to train lay users according to the patient instructions for use and explain all related safety information.

Intended patient population

The Invia Motion NPWT system is intended to be used on patients only exhibiting conditions as described in the indications for use.

The device has not been studied in pediatric patients.

Intended environment

The Invia Motion NPWT system is intended for use in acute, extended and home care settings.

When to use the device (Indications for use)

The Invia Motion Negative Pressure Wound Therapy (NPWT) system is indicated for patients who would benefit from a suction device (NPWT) as when used on open wounds it creates an environment that promotes wound healing by secondary or tertiary (delayed primary) intention by preparing the wound bed for closure, reducing edema, promoting granulation tissue formation and perfusion, and by removing exudate and infectious material.

When used on closed surgical incisions, the Invia Motion NPWT system is also intended to manage the environment of surgical incisions that continue to drain following sutured or stapled closure by maintaining a closed environment and removing exudate via the application of Negative Pressure Wound Therapy.

The Invia Motion NPWT system is appropriate for the following indications:

- Acute or subacute wounds
- Chronic wounds
- Dehisced wounds
- Pressure ulcers
- Diabetic/Neuropathic ulcers
- Venous insufficiency ulcers
- Traumatic wounds
- Partial thickness burns
- Flaps and grafts
- Closed surgical incisions

When not to use device (Contraindications)

The Invia Motion NPWT system is contraindicated in the presence of:

- Necrotic tissue with eschar present
- Untreated osteomyelitis
- Non-enteric and unexplored fistulas
- Malignancy in the wound

- Exposed vasculature
- Exposed nerves
- Exposed anastomotic site of blood vessels or bypasses
- Exposed organs

If you have concerns, speak with your healthcare professional.

! WARNING

Therapy to be performed by a healthcare professional only.

Description of the device

Check the delivery package of the Invia Motion NPWT system for completeness and general condition.

! WARNING

Do not use any other accessories than the ones described below.

- (a) 1 Invia Motion NPWT pump
- (b) 1 Invia Motion canister/tubing set 0.15l (150 ml) (sterile, single use)
- (c) 1 Invia Motion power supply
- (d) 1 Invia Motion carrying case
- (e) 1 Invia Motion Clinician Instructions for Use
- (f) 1 Invia Motion Patient Instructions for Use
- (g) 1 Invia Motion Quick Card

Display


Buttons to operate the Invia Motion NPWT pump

Conditions that affect use

⚠ CAUTION

The Invia Motion NPWT pump must remain in an upright position or lying display side up during use. Do not invert the Invia Motion NPWT pump or turn the display side down. Please see below.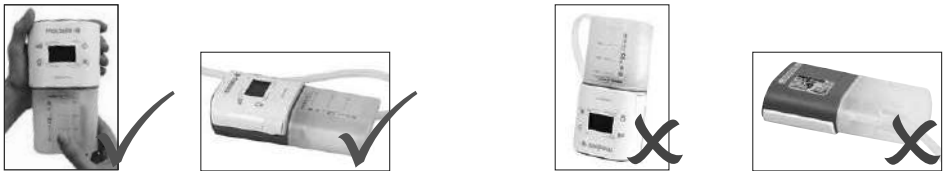

If the Invia Motion NPWT pump is placed in an inappropriate position, the pressure will be interrupted and canister full notification will appear (acoustic signal and symbol on display). The Invia Motion canister/tubing set must be replaced.

General safety at home

CAUTIONS/WARNINGS

- Do not use extension cords with the Invia Motion NPWT pump.
- Position the power cord so that it will not cause someone to trip and fall.
- Never place the power supply cord around your neck.
- Keep the power supply cord away from hot surfaces.
- Do not unplug the power supply cord from the wall outlet by pulling on the cord.
- Do not try to fix or repair the Invia Motion NPWT pump. Contact your caregiver if the Invia Motion NPWT pump is not working.
- Keep the Invia Motion NPWT pump clean and dry.
- Never place the Invia Motion NPWT pump in water or liquids.
- If the Invia Motion NPWT pump gets wet, dry with a towel. Do not dry the Invia Motion NPWT pump in a microwave.
- Keep the Invia Motion NPWT pump away from children and pets.
- Keep the Invia Motion NPWT pump protected from direct sunlight.
- Wireless communication equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, and walkie-talkies can interfere with the Invia Motion pump and should be kept at a minimum distance of 30 cm away from the Invia Motion pump. This may cause insufficient battery charging or may lead to an internal fault. See the chapter Notifications and troubleshooting for actions to take in the case of an internal fault.
- Explosion hazard – the Invia Motion NPWT pump is not for use in potentially explosive environments including oxygen enriched environments and in areas of flammable anesthetics.

If you observe a change in the vacuum level or an error condition, move the Invia Motion NPWT system at least 30 cm from the mobile phone when making or receiving calls. Turn the Invia Motion NPWT system off and on again to reset it after an error condition. Whenever possible, a safety distance of at least 30 cm between the Invia Motion NPWT pump and a mobile phone is recommended.

Sleeping

- Make sure the Invia Motion canister/tubing set will not become kinked.
- Prevent the Invia Motion NPWT pump from falling to the floor during sleep.
- Always charge the Invia Motion NPWT pump while sleeping.

Showering, bathing and washing

- Do not use the Invia Motion NPWT pump while bathing or showering.
- The Invia Motion NPWT pump can be disconnected for a short period of time (up to 20 minutes) in order to change clothes or perform personal hygiene. Discuss bathing plans with your healthcare professional.
- Prevent the Invia Motion NPWT pump from getting wet.
- Do not place the Invia Motion NPWT pump where it can fall into a shower, sink or tub.
- Do not touch the Invia Motion NPWT pump if it has fallen into water. Unplug the device immediately from the electrical outlet, disconnect the Invia Motion NPWT pump from the dressing and contact your healthcare professional.

When you leave the house

- Check the battery status and recharge the battery if necessary before leaving the house.
- For your convenience, place the Invia Motion NPWT pump into the carrying case and store excess tubing safely in the tubing pouch located in the case.
- Do not forget to take a spare canister/tubing set and the power supply with you. The power supply can be stored in the tubing pouch.

Set up instructions

WARNING

Check the Invia Motion NPWT pump frequently to make sure it is running. Visually check the wound dressing, the area around the wound and the fluid in the canister for signs of infection or other complications as instructed by your healthcare professional.

Switch Invia Motion NPWT pump on

Press

Display view when the Invia Motion NPWT pump is running. Arrow is blinking ►.

WARNING

Do not try to perform therapy without the supervision of your healthcare professional.

WARNING

NPWT must run 24 hours a day. Do not stop the therapy unless instructed by your healthcare professional or if excessive bleeding occurs.

WARNING

The dressing should be changed and therapy restarted by the healthcare professional if therapy is discontinued for more than 2 hours.

WARNING

Invia Motion NPWT pump does not detect blockage along tubing connected to the drain adapter or connected to the curved arm of the Y-connector. Check dressing tubing connected to the drain adapter or the curved arm of the Y-connector frequently throughout the day to make sure tubing is not kinked or blocked. If blockage is identified along tubing connected to the drain adapter or tubing connected to the curved arm of the Y-connector, contact your healthcare professional.

Switch Invia Motion NPWT pump off


Press the  button and hold for 3 seconds. Press only once.

Backlight

The backlight can be activated by pressing the  button. Without further interaction the backlight will turn off after 10 seconds.

Set up carrying case

The Invia Motion NPWT pump is intended to be worn in a carrying case during operation. One is included in the product package.

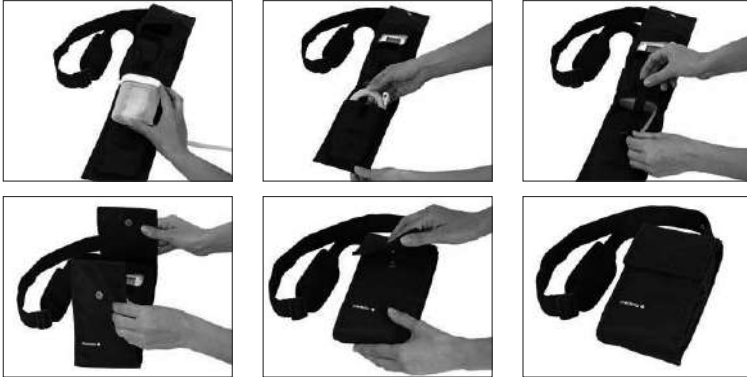1. Open the carrying case.
2. Slide the Invia Motion NPWT pump into the designated upper pocket and close the Velcro strap.
3. Place the excess tubing into the designated lower pocket and close the Velcro strap.
4. Fold the Invia Motion NPWT pump and tubing sections of the carrying case together and align the inner Velcro tabs to secure.
5. Secure the bag by closing the lid and fastening the button.

The carrying case strap can be worn over the shoulder or around the waist.

To wear over the shoulder, attach the strap clips to the carrying case. To wear as a belt, thread the strap through the two loops located on the back of the case.

Shoulder strap


Belt strap


Charge battery

Fully charge battery before first use. Charging time can be up to 4 hours. To charge the battery, please follow the instructions below.

1. Select the plug. Attach the plug to the power supply.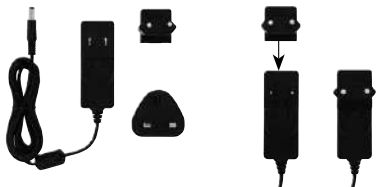

2. Open the Direct Current (DC) socket by pulling the small protective cover open.
3. Connect the small end of the power supply into the Invia Motion NPWT pump.
4. Plug the large end of the power supply into a wall outlet. The moving bars on the battery display will indicate that charging is occurring.

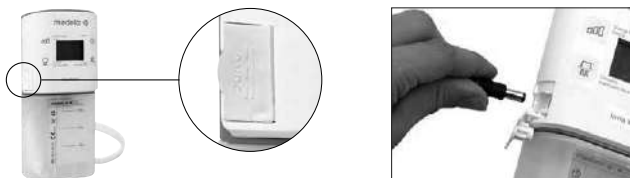Battery Status Symbols

fully charged

half full

fully discharged

Check the battery charge status periodically. Fully charge battery before first use. Charging time can be up to 4 hours. Once fully charged, the  symbol will appear constantly on the display. Charging of the battery can be slowed when exposed to high temperatures. Close the Direct Current (DC) socket by pressing the small protective cover over the opening. When the battery is fully charged, it should last approximately 10 hours. If the Invia Motion

NPWT pump is running continuously due to an air leakage, the battery may need to be recharged earlier (after approximately 4 hours).

CAUTION

NPWT therapy must be continued while charging the Invia Motion NPWT pump's battery. Do not turn the Invia Motion NPWT pump off while charging the battery.

Always charge the battery overnight. Make sure the battery compartment on the back of the Invia Motion NPWT pump is closed. Do not remove the battery from the Invia Motion NPWT pump.

Change canister/tubing set

 To be used by healthcare professionals or adequately trained caregivers only.

Change the canister minimum once a week or when it appears full, when the canister full signal sounds or when the canister full symbol  is displayed.

1. Wash hands and prepare clean paper towel.
2. Remove the Invia Motion NPWT pump from the carrying case. Switch the Invia Motion NPWT pump off by pressing the  button and holding for 3 seconds. Press only once.

- Position the clamp next to the quick-connector and close the clamp on the pump tubing and close the clamp on the dressing tubing.

Clamp open

Clamp closed

- Disconnect the Invia Motion NPWT pump tubing from the dressing tubing by pressing on the sides of the quick-connector. Have a paper towel or tissue available to cover the tip of the used tubing to protect against wound fluid leakage.

- Detach the full canister/tubing set by rotating the canister to the left.

- Wash hands.
- Obtain a new Invia Motion canister/tubing set and open the packaging.


CAUTION

Do not use an Invia Motion canister/tubing set if the sterile packaging is damaged or has expired.

 Make sure the correct canister is used: **REF** 087.4016.
Contact your healthcare professional if the correct canister is not available.

8. Remove the new canister/tubing set from the packaging and attach it to the Invia Motion NPWT pump by pressing the canister to the Invia Motion NPWT pump (1) and rotating to the right (2). Make sure the canister is aligned with the label to the front.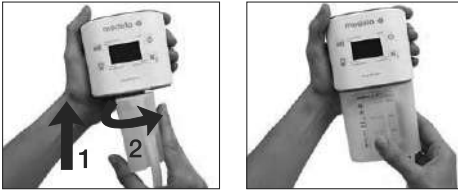

9. Connect the Invia Motion tubing connector to the wound drain tubing. Push together until you hear a click.

10. Open the clamp on the dressing tubing. Also make sure the clamp on the pump tubing is open.

11. Switch the Invia Motion NPWT pump on by pressing the  button. Slide the Invia Motion NPWT pump into the carrying case.

12. Discard the used canister/tubing set according to instructions provided by your healthcare professional.

Notifications and troubleshooting

Notifi- cations	Notification description	Acoustic signal	Trouble- shooting
	Leakage	Yes. Can be muted by pressing	See page 22
	Blockage	Yes. Can be muted by pressing	See page 22
	Canister full	Yes. Can be muted by pressing	See pages 20–21
	Battery low (symbol is blinking)	Yes. Can be muted permanently by pressing	See page 23
	Battery fully discharged (symbol is blinking)	Yes. Can be muted by switching off the pump.	See page 24
	Battery missing (symbol is blinking)	Yes. Can be muted by pressing	See page 25
	Internal fault	Yes. Can not be muted	See page 25
	Pump time expired (symbol is blinking) (Example for Invia Motion – 15 Days)	No	See page 25

Do not panic if an acoustic signal sounds.

Follow the instructions below.

1. If the Invia Motion NPWT pump is in the carrying case open the lid of your carrying case.

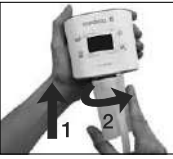
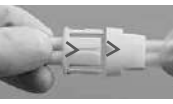
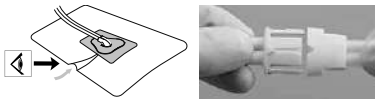
2. Press the button to mute the acoustic signal for 60 seconds (the button can be pressed every time the signal sounds, with the exception of “Battery fully discharged” and after the timer of 15 or 60 days plus 72 hours of safety time is used up).

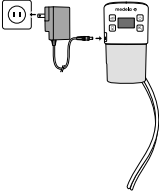
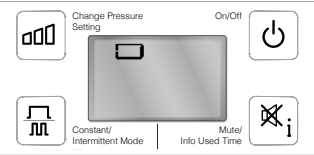
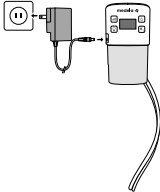
3. Check the display for one of the following visual symbols.

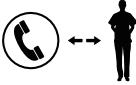
Visual symbol displayed	What does this symbol mean?	What do I have to do?
 <p>Blinking! + acoustic signal!</p>	<p>The canister is full</p>	<p>Mute the acoustic signal by pressing the  button.</p> <ol style="list-style-type: none"> 1. Wash hands and prepare clean paper towel. 2. Remove the Invia Motion NPWT pump from the carrying case. Switch the Invia Motion NPWT pump off by pressing and holding the  button for 3 seconds. Press only once. 3. Position the clamp next to the quick-connector and close the clamp on the pump tubing and close the clamp on the dressing tubing. <div style="display: flex; justify-content: space-around; align-items: center; margin: 10px 0;">   </div> <p style="display: flex; justify-content: space-around; margin: 0 0 10px 0;"> Clamp open Clamp closed </p> 4. Disconnect the Invia Motion NPWT pump tubing from the dressing tubing by pressing on the sides of the quick-connector. <div style="display: flex; justify-content: center; align-items: center; margin: 10px 0;">   </div> <p>Have a paper towel or tissue available to cover the tip of the used tubing to protect against wound fluid leakage.</p> 5. Detach the full canister/tubing set by rotating the canister to the left. <div style="display: flex; justify-content: space-around; align-items: center; margin: 10px 0;">    </div> <p>Discard the used canister/tubing set according to instructions provided by your healthcare professional. →</p>

Visual symbol displayed	What does this symbol mean?	What do I have to do?
 <p>Blinking! + acoustic signal!</p>	<p>The canister is full</p>	<ol style="list-style-type: none"> 6. Wash hands. 7. Obtain and open a new Invia Motion canister/tubing set package. <p>Caution  Do not use an Invia Motion canister/tubing set if the sterile packaging is damaged or has expired.</p> <p> Make sure the correct canister is used: 087.4016. Contact your healthcare professional if the correct canister is not available.</p> <ol style="list-style-type: none"> 8. Remove the canister/tubing set from the packaging and attach it to Invia Motion NPWT pump. → <div style="display: flex; justify-content: space-around;">   </div> <ol style="list-style-type: none"> 9. Connect the Invia Motion NPWT pump tubing to the dressing tubing. Push together until you hear a click. <div style="display: flex; justify-content: space-around;">   </div> <ol style="list-style-type: none"> 10. Open the clamp on the dressing tubing. Also make sure the clamp on the pump tubing is open. 11. Switch the Invia Motion NPWT pump on by pressing the  button. Slide the Invia Motion NPWT pump into the carrying case. 12. Ask your clinician how to properly dispose of a full canister/tubing set. <p>If you are unable to change the canister contact your healthcare professional.</p>

Visual symbol displayed	What does this symbol mean?	What do I have to do?
 <p>Blinking! + acoustic signal!</p>	<p>There is a blockage present</p>	<p>Mute the acoustic signal by pressing the  button.</p> <p>Check the canister tubing and the dressing tubing to ensure tubing is not twisted, kinked or clamped. Verify that the correct canister is used: REF 087.4016. If the used canister is not the correct one contact your healthcare professional.</p> <p>The notification symbol and the acoustic signal may still be active, even if situation is resolved. Wait for some time to let the pump react.</p> <p>If you are unable to eliminate the blockage, switch the Invia Motion NPWT pump off by pressing the  button and holding for 3 seconds. Contact your healthcare professional for instructions.</p>
 <p>Blinking! + acoustic signal!</p>	<p>There is a leak present</p>	<p>Mute the acoustic signal by pressing the  button.</p> <p>Check the dressing for leaks and pat the transparent film (clear bandage) placed over the wound and surrounding skin. Make sure the Invia Motion tubing quick-connector is fully engaged.</p> <div data-bbox="426 1034 801 1134">  </div> <p>The notification symbol and the acoustic signal may still be active, even if situation is resolved. Wait for some time to let the pump react.</p> <p>If you are unable to eliminate the leak, switch the Invia Motion NPWT pump off by pressing the  button and holding for 3 seconds. Contact your healthcare professional for instructions.</p>

Visual symbol displayed	What does this symbol mean?	What do I have to do?
 <p>Blinking! + acoustic signal!</p>	<p>The battery is low</p>	 <p>Mute the acoustic signal by pressing the  button. The battery requires charging. Connect the Invia Motion NPWT pump to the supplied power supply and power source and fully charge battery. Charging time can be up to 4 hours. Moving bars on the battery display will indicate that charging is occurring. Do not turn the Invia Motion NPWT pump off while charging.</p> 

Visual symbol displayed	What does this symbol mean?	What do I have to do?
 Blinking! + acoustic signal!	The battery is fully discharged	<p>The negative pressure is no longer maintained. When the pump is not connected to the power supply within 10 minutes, it switches off.</p>  <p>Charge the battery immediately. Connect the Invia Motion NPWT pump to the supplied power supply and power source. Moving bars on the battery display will indicate that charging is occurring.</p>  <p>Switch the Invia Motion NPWT pump on by pressing the  button. If you do not have access to electricity and the battery is fully discharged, contact your healthcare professional for instructions.</p>

Visual symbol displayed	What does this symbol mean?	What do I have to do?
 Blinking! (Example of 15 days version)	Available operation time elapsed	Contact your healthcare professional. Do not turn the Invia Motion NPWT pump off! 
 Blinking! + acoustic signal!	Battery missing	Ensure that the battery compartment cover on the backside of the Invia Motion NPWT pump is mounted and closed properly. Do not remove the battery.
 Acoustic signal!	Internal fault	Turn the Invia Motion NPWT pump off by pressing the  button and holding for 3 seconds.  Contact your healthcare professional immediately for further instructions.

If you are unable to identify any symbol on the display, switch off the Invia Motion NPWT pump by pressing the  button and holding for 3 seconds. Contact your healthcare professional.

 **WARNING**

If the Invia Motion NPWT pump fails or cannot be switched on, call your healthcare professional immediately.

Info used time

REF 087.4022 Invia Motion – 15 Days Negative Pressure Wound Therapy system


REF 087.4024 Invia Motion – 60 Days Negative Pressure Wound Therapy system

These Invia Motion NPWT pump versions have a runtime of 15 and 60 days.

Press the  button to obtain information regarding the treatment time used.

The picture below illustrates a 15 days runtime Invia Motion NPWT pump that has operated for 6 days and 23 hours. The remaining runtime of this Invia Motion NPWT pump would be 8 days and 1 hour.

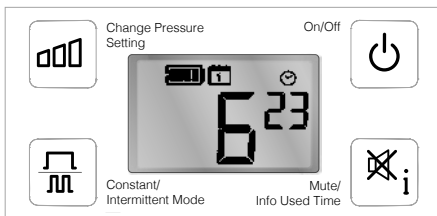Please inform your healthcare professional when the remaining operation time is three days.

Cleaning/Disinfection

WARNING

Unplug the pump before cleaning. Close the small protective cover on the DC socket.

The Invia Motion NPWT pump can be wiped with a damp cloth using a mild household cleaner. Invia Motion can be disinfected with the disinfecting agent group “alcohol”.

WARNING

Reuse on another patient might cause cross contamination.

Warranty

The warranty period for the Invia Motion NPWT system expires after 15 or 60 days of usage plus 72 hours of safety time, if used in accordance with the instructions for use. The manufacturer is not liable for any damage or consequential damage caused by incorrect operation, inappropriate usage or use by unauthorized persons.

Maintenance

The Invia Motion NPWT pump is maintenance free and will not require service.

If an Invia Motion NPWT pump fails within the warranty period due to a manufacturing defect, it will be replaced. The original Invia Motion NPWT pump will need to be returned to the supplier.

Disposal

Ask your healthcare professional how to dispose of used canister/tubing sets and the Invia Motion NPWT pump at the end of your therapy.

Technical specifications

Accessories storage conditions

Accessories must remain in the packaging and be stored within a temperature range of -20° Celsius to $+50^{\circ}$ Celsius.

Operation environmental conditions

The Invia Motion NPWT pump and accessories must be operated within a temperature range of $+5^{\circ}$ Celsius to $+40^{\circ}$ Celsius. Do not operate the product in extreme cold or heat. Avoid high relative humidity (15–93% is permissible).

CAUTION

Do not operate the Invia Motion NPWT pump above 3000 meter above sea level.

Protection Class IP22

The Invia Motion NPWT pump is protected against ingress of solids larger than 12.5 mm and dripping water.

IP22

Signs and symbols

 This symbol indicates a CAUTION or WARNING associated with the device (see page 4).

 This symbol indicates a class II device.

 This symbol indicates not to dispose of the device with household refuse (for EU only).

 This symbol indicates the date of manufacture.

 This symbol indicates the name and the address of the manufacturer.

 This symbol indicates the device is sterilized using ethylene oxide.

 This symbol indicates the device is MR unsafe.

 The symbol indicates the item is a medical device.

 This symbol indicates that this is a Prescription Device. CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician (for US only).

 This symbol indicates a type BF applied part.

 This symbol indicates the manufacturer's catalog number.

 This symbol indicates the manufacturer's serial number.

 This symbol indicates the manufacturer's batch code.

IP22 This symbol indicates the device has protection against the ingress of dripping water and solid foreign objects of 12.5 mm and greater.

 This symbol indicates that the device should not be used after the date shown.

This symbol indicates to follow instructions for use.

This symbol indicates the compliance with additional USA and Canada safety requirements for medical electrical equipment.

This symbol indicates not to use the device if package is damaged.

pcs

This symbol indicates the number of items.

This symbol indicates the Direct Current socket.

This symbol indicates a single use device. Do not reuse the device.

This symbol indicates the temperature limitation for operation, transport and storage.

This symbol indicates the humidity limitation for operation, transport and storage.

This symbol indicates the atmospheric pressure limitation for operation, transport and storage.

This symbol indicates to keep the device dry.

This symbol indicates to handle the fragile device with care.

This symbol indicates to keep the device away from sunlight.

This symbol indicates that the marked item or its material is part of a recovery or recycling process.

This symbol indicates the correct upright position of the transport package.

This symbol indicates that the device is in conformance with the Medical Device Directive 93/42/EEC.

Electromagnetic compatibility (EMC)

Invia Motion is EMC-tested in conformity with the requirements of IEC 60601-1-2:2007 and IEC 60601-1-2:2014 4th Edition according clause 7 and 8.9.

WARNING

The Invia Motion pump should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the Invia Motion pump should be observed to verify normal operation in the configuration in which it will be used.

WARNING

Do not use other accessories than those specified or sold by the manufacturer as replacement parts for internal components as it may result in increased emissions or decreased immunity of the Invia Motion pump.

WARNING

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Invia Motion. Otherwise, degradation of the performance of this equipment could result.

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Medela AG
Medela UK Ltd.